STOMOCUR® Ostomy Belt

	Instructions for use	<i>EN</i>
---	-----------------------------	-----------

Please read the instructions for use carefully, as they contain important information on how to use your STOMOCUR® accessories.
Intended purpose: The ostomy belt is designed for people with an artificial intestinal outlet. It is used to provide an additional fixing point for the STOMOCUR® supply system on the body.
Using the ostomy belt: - Adjust the ostomy belt to your waist circumference. - Carefully stretch the ostomy belt to connect the hook of the belt with the eyelet on the supply system. - Finally, straighten the belt. - The ostomy belt should be washed after repeated use. (Hand wash at 30 degrees)

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Warnings

- Please only use the STOMOCUR® product indicated in these instructions for its intended purpose (see 'Intended purpose'). - STOMOCUR® products must not be disposed of in the WC. Please dispose of them in the correct way. - If you have any questions or problems during use, please contact your specialist doctor or your stoma therapist. - If a serious injury (incident) occurs in connection with the use of this medical device, please contact your local distributor or the manufacturer, as well as your competent local authority. - Contains viscose, rubber-coated elastane and cotton.
Storage conditions: Store in a cool, dry place away from light.

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 中华人民共和国 / People's Republic of China	 Guangxi Sibeide Medical Co. Ltd.
 F4&F5, Building 1	 Hongxiang Standard Factory, No.2
 Hongli Road	 Nanning, Guangxi
 People's Republic of China	 +86) 771 48 75 549
 gxzz@vip.163.com	

 Shqipëria / Albania	 Mega Store shpk Tirane
 Lagjja Katund I Ri, Njesia Administrative Kashar	 Rr “Sadik Staveleci”, Godina Nr.16
 1051 Kashar, Tirane	 Shqipëria
 +355 68 20 87 993	 medicamenta@medicamenta.al
 www.medicamenta.al	

 United Kingdom	 Unora Healthcare Limited
 Unit B, The I O Centre	 Barn Way
 Lodge Farm	 Industrial Estate
 Northampton NN5 7UW	 United Kingdom
 +44 (0) 1604 586 529	 www.unora.co.uk

 Danmark / Denmark	 OneMed A/S
 Kappa 3	 8382 Hinnerup
 Denmark	 +45 86 109 109
 info.dk@onemed.com	

 România / Romania	 PHARMIA TELNET S.R.L.
 Bd. Maresal Averescu Alexandru nr.13,	 Bl.7, Sc.B, Ap.12, Sector 1, Bucuresti, 011454
 România	 +40 (0) 21 250 06 20
 office@pharmatelnr.ro	 www.pharmatelnr.ro

 Polska / Poland	 Pofam Poznań Sp.z o.o.
 ul. Lindego 6	 60-573 Poznań
 Polska	 +48 (0) 61 845 69 00
 office@pofam.poznan.pl	 www.pofam.poznan.pl

 Österreich / Austria	 Quadramed
 Quadramed-Haas OG	 office@quadramed.at
 www.quadramed.at	

 香港 / Hong Kong	 Solutions Health Care Products & Services Ltd.
 Rm. 1104-05, 11/F., Capitol Centre	 5-19 Jardine's Bazaar
 Causeway Bay	 Hong Kong
 +852 2273 8233	 cs@solutions-healthcare.com.hk
 eshop.solutions-healthcare.com.hk	

 Italia / Italy	 SVAS BIOSANA S.P.A.
 Via Trentola, 7	 80049 Somma, Vesuviana (Na)
 Italia	 +39 (0) 08 18 99 54 11
 svas.biosana@svas.it	 www.svas.it

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