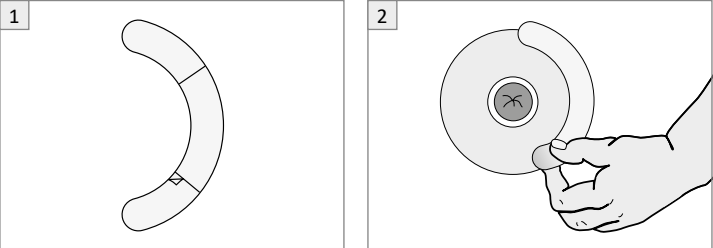
STOMOCUR® Fixierstreifen

Instructions for use

EN

Please read the instructions for use carefully, as they contain important information on how to use your STOMOCUR® products.

Elastic fixing strip for stoma care

Intended purpose:
The STOMOCUR® fixing strip is used as an additional adhesive area on the edge of the stoma base plate.
Application of the fixing strip:
1. Remove the protective film.
2. Fix the strip in the desired position on the skin protection plate of your stoma. When doing so, make sure that the fixing strip covers part of the skin protection.
3. Apply the fixing strip smoothly over the skin.
4. Removal: The fixing strip can be pulled off with the skin protection plate and disposed of as household waste.

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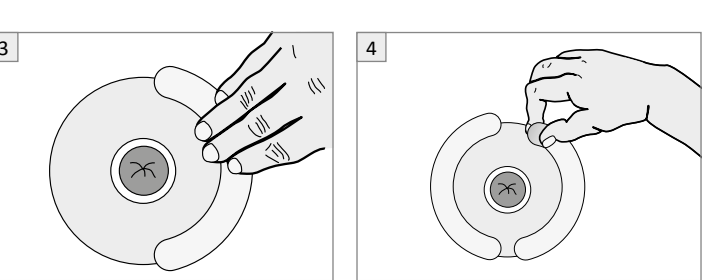
Warnings

- Please use the **STOMOCUR® accessories** indicated in these instructions for the respective purpose (see Intended purpose).
- Do not use this product for any other purpose.
- The **STOMOCUR® accessories** specified here must not be used if the product is damaged.
- If you have any questions or problems in using the product, please contact your specialist or your stoma therapist.
- If a serious injury (incident) occurs in connection with the use of this medical device, please contact your local distributor or the manufacturer, as well as your competent local authority.

Storage conditions

- Please store the **STOMOCUR® accessories** in a cool, dry place away from light.

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Mediplast AB Bronsåldersgatan
Unora Healthcare Limited