STOMOCUR® SGX - Gelling Sachets

Instructions for use

EN

Please read the instructions for use carefully, as they contain important information on how to use your STOMOCUR® accessories.

Intended purpose:
STOMOCUR® SGX - Gelling Sachets are designed for people with an artificial intestinal outlet. **SGX - Gelling Sachets** serve to convert thin fluid excretions into a solid gel to make it easier to drain the pouch.

Using STOMOCUR® SGX:
1. Add a **SGX - Gelling Sachet** into the pouch when putting it on or after emptying it.
2. One sachet will thicken up to 400 ml of fluid excretion, binding it immediately and making it easier to empty the pouch.
3. The pouch can be emptied into the toilet as usual.

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FORLIFE

Warnings

• Please only use the STOMOCUR® product indicated in these instructions for its intended purpose (see 'Intended purpose').
• **SGX - Gelling Sachets** are not for consumption. The sachets are not toxic, but they should be kept out of reach of children.
• If the contents of the **SGX - Gelling Sachets** get into your eyes, rinse with clean water.
• If you have any questions or problems during use, please contact your specialist doctor or your stoma therapist.
• If a serious injury (incident) occurs in connection with the use of **SGX - Gelling Sachets**, please contact your local distributor or the manufacturer.

Storage conditions:
• Store the **STOMOCUR® SGX** in a cool, dry place away from light.

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 中华人民共和国 / People's Republic of China
Guangxi Sibeide Medical Co. Ltd. F4&F5, Building 1 Hongxiang Standard Factory, No.2 Hongli Road Nanning, Guangxi People's Republic of China +(86) 771 48 75 549 gxzz@vip.163.com

 Shqiperia / Albania
Mega Store shpk Tirane Lagjia Katund i Ri, Njesia Administrative Kashar Rr “Sadik Stavileci”, Godina Nr.16 1051 Kashar, Tirane Shqiperia +355 68 20 87 993 medicamenta@medicamenta.al www.medicamenta.al

 United Kingdom
Unora Healthcare Limited Unit B, The I O Centre Barn Way Lodge Farm Industrial Estate Northampton NN5 7UW United Kingdom +44 (0) 1604 586 529 www.unora.co.uk

 Danmark / Denmark
OneMed A/S Kappa 3 8382 Hinnerup Denmark +45 86 109 109 info.dk@onemed.com

 România / Romania
PHARMA TELNET S.R.L. Bd. Maresal Averescu Alexandru nr.13, Bl.7, Sc.B, Ap.12, Sector 1, Bucuresti, 011454 România +40 (0) 21 250 06 20 office@pharmatelnr.ro www.pharmatelnr.ro

 Polska / Poland
Pofam Poznań Sp.z o.o. ul. Lindego 6 60-573 Poznań Polska +48 (0) 61 845 69 00 office@pofam.poznan.pl www.pofam.poznan.pl

 Österreich / Austria
Quadramed Quadramed-Haas OG office@quadramed.at www.quadramed.at

 香港 / Hong Kong
Solutions Health Care Products & Services Ltd. Rm. 1104-05, 11/F., Capitol Centre 5-19 Jardine's Bazaar Causeway Bay Hong Kong +852 2273 8233 cs@solutions-healthcare.com.hk eshop.solutions-healthcare.com.hk

 Italia / Italy
SVAS BIOSANA S.P.A. Via Trentola, 7 80049 Somma, Vesuviana (Na) Italia

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